Tooth Whitening System for direct application

Cosmetic 0,1% HP

ELITEsmile Whitening Pen is the modern and simple method of tooth whitening to go. The specially developed formulation with 0,25% carbamide peroxide (equals 0,1% hydrogen peroxide) with high viscosity provides the basis for an effective, tooth-friendly application. Sensitivities are minimized. For more information on professional tooth whitening visit www.ELITEsmile.se

Intended use

Whitening Pen for whitening teeth with direct application of the gel to the tooth surface. Prosthetic treatments, such as inlays, crowns or implants cannot be altered in color.

Instructions for use: How to achieve maximum results.

- Brush and floss your teeth!
- Take off the closing cap and start twisting the back of the Whitening Pen. Twist slowly to allow the gel to move to the brush tip.
- Apply the gel with the brush tooth for tooth in a thin layer directly on the front surface of the teeth.
Do not apply to the gingiva or soft tissue.
- Let the gel for 20 minutes in mouth for whitening.
Rinse mouth with warm water and brush teeth – ready!
- Do not smoke during and for 30 minutes after using of the product.
- For most effective results please refrain from consuming tea, coffee, red wine, cigarettes etc. for at least 2 hours after whitening.

Tip

For daily care of white teeth the ELITEsmile Whitening Toothpaste shows best results.

Possible side effects

At intended use no side effects are expected. If tooth sensitivity or gum irritation occur interrupt treatment until symptoms are gone. Symptoms are temporary and should subside a day following termination of product use.

Warning

Avoid contact with eyes. Rinse immediately if product comes into contact with them.

Storage

Store in a cool, dark place. Do not freeze. Keep out of the reach of children.

Ingredients Whitening Pen

Glycerin, Aqua, Carbamide Peroxide, Polyethylene Glycol, Triethanolamine, Carbomer, Sodium Saccharin, EDTA, Aroma.

For shelf-life and LOT see pen and carton box.

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